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SINUS:

- 1- Split Master 3
 - 2- Bone Expander Kit
 - 3- Concave Bone Expander Kit
 - 4-Bone Crester Kit
 - 5-Saw Cover Kit 2
 - 6-Split Chisel & Ochsenbein Chisel
 - 7-Bone Well & Mallet
-

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SPLITTING:

- 1- Masi Drill Kit 3
 - 2- Sinus Master Kit 3
 - 3- Hydro Lift System
 - 4-Osteotome-Rigid / Thermos
 - 5-Sinus Lift & Sinus Lift Crestal
-

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RESCUE:

- 1- Fixture & Screw Rescue Master
 - 2- Fixture & Screw Removal Kit
 - 3-Tissue Punch & Trepine Bur & Saw
 - 4- Tissue Punch
 - 5- Trepine Bur
 - 6-M-toque Driver Kit
 - 7-Implant Guide Kit
-

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Orthodontists:

- 1- Orthodontic Screw Set
 - 2- Orthodontic Tool Set
 - 3- Orthodontic Plate Set
-

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GBR KIT Membrane Fixed Screw With Membrane Tac Combo Kit

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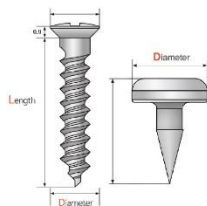
لینک سایت : https://www.mctbio.com/GBR/TentingScrew_set.html

GBR-KIT III (Screw 30EA Bone Tac :18EA)

Membrane fixation screw with membrane tac combo kit.

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GBR KIT Membrane Fixation Screw With Membrane Tac Combo Kit

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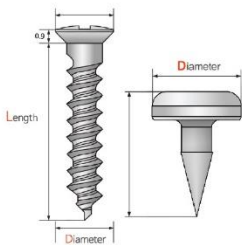
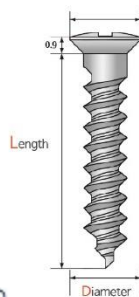
For GBR

Membrane Fixed Screw Set (Screw 54EA)

Consist of short length screws (3 & 4 mm) for membrane fixation (Ti-mesh).

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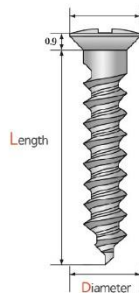
لینک سایت: https://www.mctbio.com/GBR/BlockBoneScrew_set.html

Block Bone Screw Set (Screw 54EA)

Consist of long length screws (8, 10, 12mm) for block bone fixation or Tenting purpose.

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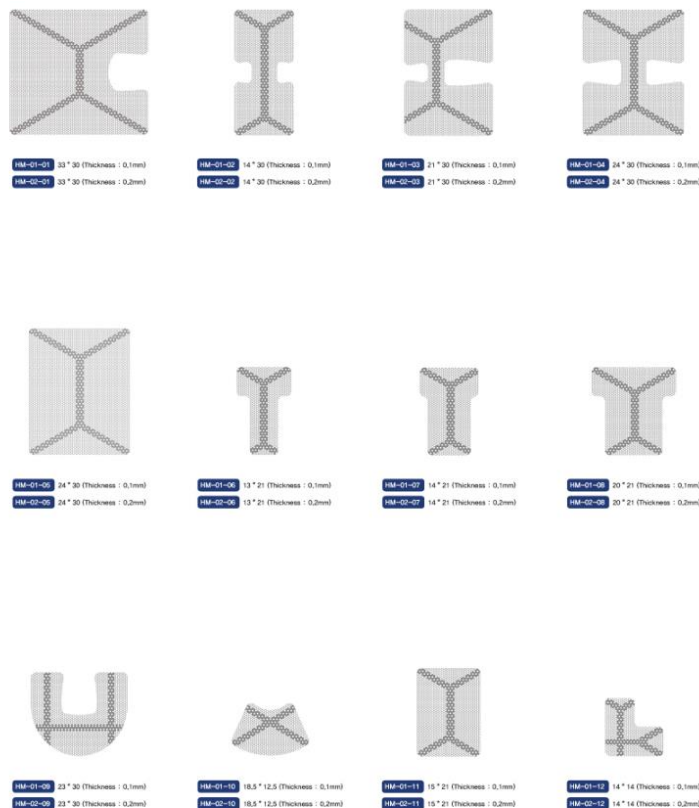
7-PRF Box

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PRF-BOX

(Fibrin Compressor – Plate / Rich / Fibrin)

Character of PRF BOX

1. Prevent infection by not using gauze.
2. Growth Factor collected by PRF BOX and used in Bone Grafting is helpful in the formation of new bone.

Lid of Case
PRF-01-03

Press Board
PRF-01-01

PRF-BOX
PRF-BOX

Bottom Board & Case
PRF-01-02 PRF-01-04

Direction for Use

1. Draw 5cc blood from patients.
2. Extract PRF Gel by centrifuging the blood sample.
3. Put the yellow part of PRF Gel into the PRF Box and press by press.
4. Use PRF membrane which has been separated from the liquid.

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BCM2-01 New Bone Chip Maker

Detail View

Bone collection greatly increased to more than 30%

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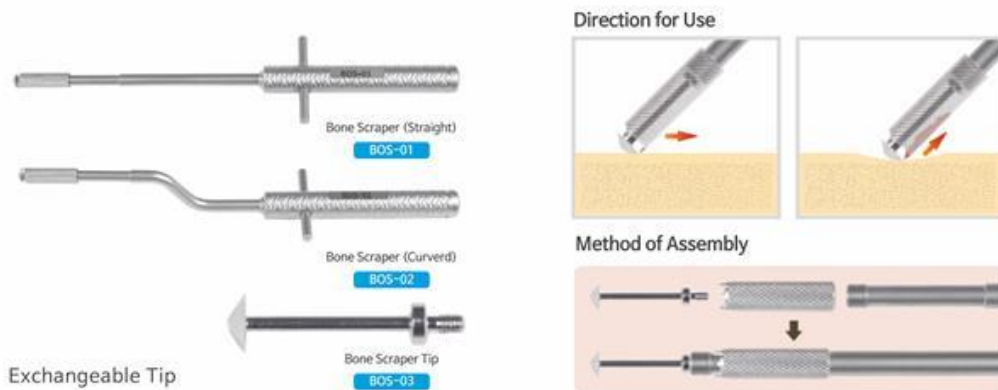
Bone Scraper

It Scratches bone to collect and be sterilized as well as economically used with replaceable tip

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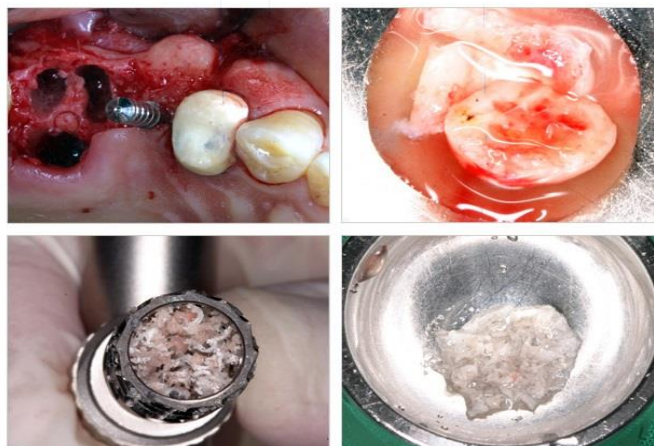
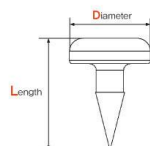
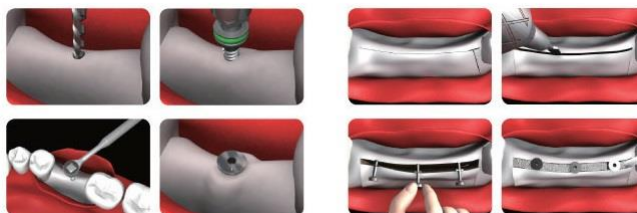
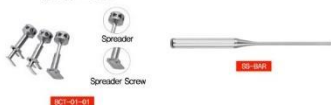
Bone Carrier

Split Chisel

4mm Straight SC-01
4mm Curved SC-02
6mm Straight SC-03
6mm Curved SC-04

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SBE-01 Bone Expander

Bone Expander

It used for single implant with narrow bone width or weak bone to expand bone or upgrade bone.

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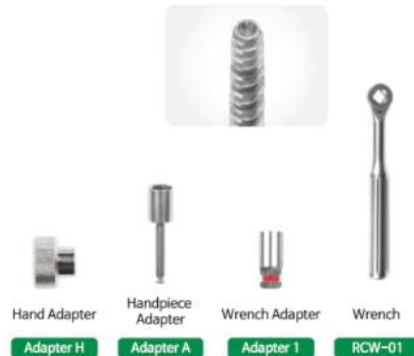
BCT-01 Bone Crester

Bone Crester

It maintains the initial fixing power of implant to increase success rate of implant without long stretched ridge when multi-implanting with narrow bone width.

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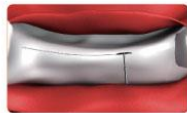
Split Chisel

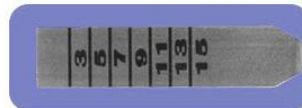
SPC-01 / SPC-02 / SPC-03 / SPC-04 / SPC-05

Characteristic of Split Chisel : It is 0.5mm thick used to widen narrow area in surgery.

Size of Split Chisel

Detail View

7-Bone Well & Mallet

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Implant Instrument

Bone Well

BOW-01 / BOW-02 / BOW-03 / BOW-04

Characteristic of Product

Bone Well used for bone mixing, Bone Well Lid prevents bone dry and contaminations during surgery time.

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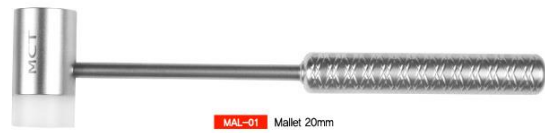
Masai Drill IV (Magnetic Sinus Artificial Intelligence Drill) MAS4-01

Composition of MAS4-01

MAS4-01 Masai Drill IV

Magnetic Drill

It is magnetic drill which delete maxillary bone without damage of sinus membrane. Using French Magnet which can beverse with 400°C, magnetic of drill is not change with subacute temperature.

Character of Magnetic Drill – Three Kinds Safety devices

1. Blade of drill have groove in the diamond bar, safety and cutting force is excellent, and even though drill reach sinus membrane, membrane is not damaged.
2. Masai drill are tripod against sill which made by initial drill, and even if pressed at moment of bone punching, masai drill did not go through into the hole.
3. At the moment of bone punching, upper part of masai drill was separate from the Lower part by magnet force and stop operate. It can perceive by hand, also can confirm through separation of gear.

Component

Initial Drill

Stopper

Aqua Tip

- Aqua Tip
1. After marking hole with Masai drill use aqua tip to fit up the membrane.
 2. Engage the Aqua tip onto the hole.
 3. Fill the syringe with saline water and join with hose.
 4. Place the syringe into the roller and stick the hose with Aqua tip.
 5. To fit up the membrane, turn the roller.

Direction for Use

2- Sinus Master Kit 3

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Sinus Master III

SMK2-01

Composition of SMK3-01

SMK3-01 Sinus Master III

For Lateral and Crestal Sinus Lifting

1. A lateral Maxillary window can be created efficiently with the Lateral Drills,
 2. For Crestal approach, First use the Crest drill with appropriate depth stop until the last 1mm,
 3. Finish the last 1mm with Crest Reamer with 1mm depth stop attached,
- The Crest Reamer is made of diamond particles to avoid snagging the membrane,
4. Use sinus lift instruments to lift up the sinus membrane,

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WSL-01 Hydro Lift System

Aqua Tip

1. Maxillary sinus membrane can be easily by hydrostatic pressure.
2. Various size of aqua tip(2.8, 3.0, 3.3, 3.5, 3.8, 4.0)
3. Compatible to a variety of maxillary sinus elevation drill.
4. Sanitary membrane elevation due to use of a disposable syringe
5. Because treatment of creastal approach is possible within 3mm in residual bone, the time and cost is reduced.
6. Desired amount of bone can be easily implanter.

Hydro Lift System

WSL-01-01

WSL-01-02

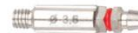
WSL-01-2.8

WSL-01-3.0

WSL-01-3.3


WSL-01-3.6

WSL-01-3.8

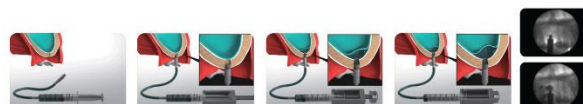WSL-01-4.0

4-Osteotome-Rigid / Thermos

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Implant Instrument

Osteotome – Rigid, Thermos, Gaori

OSR-SET / OST-SET / OSG-SET

Characteristic of Osteotome

Detail of Rigid Set

Detail of Thermos Set

Detail of Gaori Set

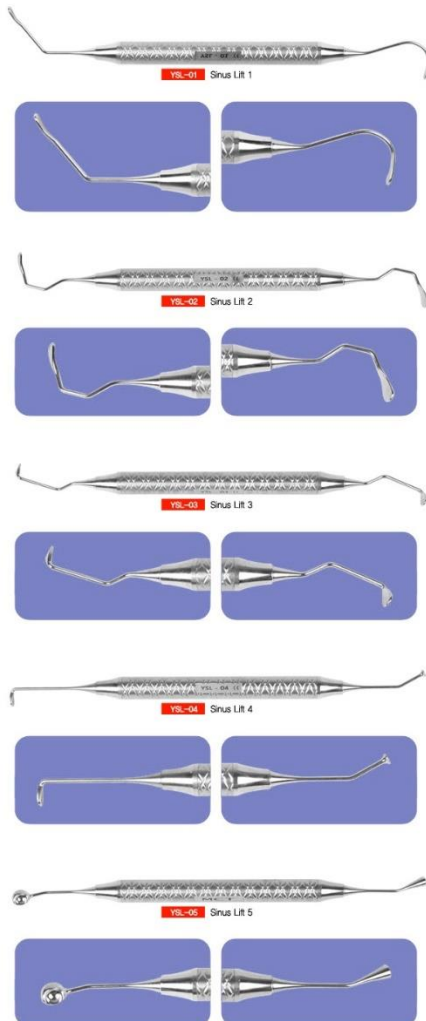
5-Sinus Lift & Sinus Lift Crestal

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General Instrument Sinus Lift YSL-01 / YSL-02 / YSL-03 / YSL-04 / YSL-05

Detail View

General Instrument Sinus Lift Crestal

SLC-01 / SLC-02 / SLC-03 / SLC-04

Detail View

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RESCUE

1- Fixture & Screw Rescue Master

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Screw Remover Kit

SRK-01

Composition of SRK-01

SRK-01 Screw Remover Kit

Fixture Remover Kit

FRK-01

Composition of FRK-01

FRK-01 Fixture Remover Kit

Screw Remover Kit

1. Pick certain guide for used abutment.
2. Attach the guide and hold it with guide holder.(SRG-H)
3. Use screw remover drill to make hold in the core of broken screw in reverse way.
4. Use Tap drill to remove the broken screw in reverse way.

FRD-01(3EA)

Fixture Remover Kit

1. Excellent combination of Remover and Trophine Bar
2. Trophine bar that can remove implant without bone loss.
3. One remover applicable to all sizes of implants
4. Solid durability

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Fixture & Screw Removal Master (Rescue Master Kit)

Fixture & Screw
Removal Master

F5Rk-02

Fixture & Screw Removal Master
(Rescue Master Kit)

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TAT-02

Composition of TAT-02

TAT-02 Tissue Punch & Trephine Bur & Saw Set

Tissue Punch

Ø4.0 TIP-4,0
Ø4.5 TIP-4,5
Ø5.0 TIP-5,0
Ø5.5 TIP-5,5

Trephine Bur

Ø5.0 TPL-05
Ø6.0 TPL-06
Ø7.0 TPL-07
Ø8.0 TPL-08

Saw

Ø7 SAW-01
Ø10 SAW-02

4- Tissue Punch

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Implant Instrument

Tissue Punch

TIP-3.5 / TIP-4.0 / TIP-4.5 / TIP-5.0 / TIP-5.5 / TTI-4.0 / TTI-5.0

Issue Punch

Ø3.5

TIP-3.5

Ø4.0

TIP-4.0

Ø4.5

TIP-4.5

Ø5.0

TIP-5.0

Ø5.5

TIP-5.5

Issue Punch (Special Type)

Ø4.0

TTI-4.0

Ø5.0

TTI-5.0

5- Trepine Bur

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Implant Instrument

Trepine Bur

TPB-3 / TPB-4 / TPB-5 / TPB-6 / TPB-7 / TPB-8
TPL-3 / TPL-4 / TPLB-5 / TPL-6 / TPL-7 / TPL-8

■ Trepine Bur

Ø3
TPB-3

Ø4
TPB-4

Ø5
TPB-5

Ø6
TPB-6

Ø7
TPB-7

Ø8
TPB-8

■ Trepine Bur Long type

Ø3
TPL-3

Ø4
TPL-4

Ø5
TPL-5

Ø6
TPL-6

Ø7
TPL-7

Ø8
TPL-8

6-M-toque Driver Kit

- Great for hard to reach areas near molars
- Adjustable Torque between 10-40Ncm
- Accepts all Standard Contra Angle Drivers
- Simply spin the back section of the handle to select the torque value, then twist the whole handle to torque
- Two piece construction: Torque handle and handpiece attachment
- Includes a 0.050 Hex Driver

TH-01

TD-01

TH-01

TD-01

7-Implant Guide Kit

- Allows a mesio-distal guidance when placing implants
- Components
 - 5 Guides
 - 5 Posts
 - 4 Drills
- Standard Kit Components
 - Guides
 - (1) 6mm Guide
 - (1) 7.5mm Guide
 - (1) 9mm Guide
 - (1) 10.5mm Guide
 - (1) 12mm Guide
 - Posts
 - (1) 6mm Post
 - (1) 7.5mm Post
 - (1) 9mm Post
 - (1) 10.5mm Post
 - (1) 12mm Post
 - Drills
 - (1) 4mm Drill
 - (1) 5mm Drill
 - (1) 8mm Drill
 - (1) 10mm Drill

Implant Guide Kit

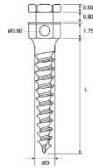
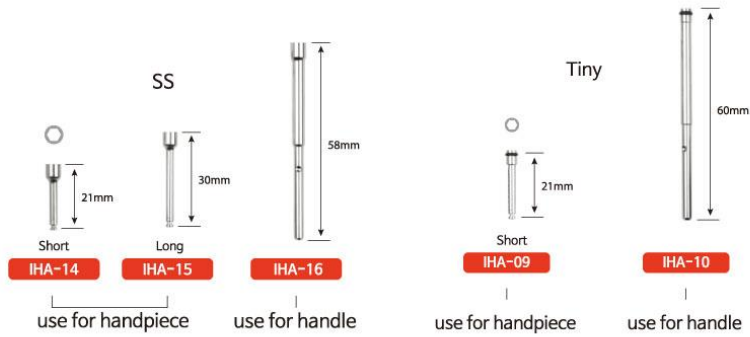
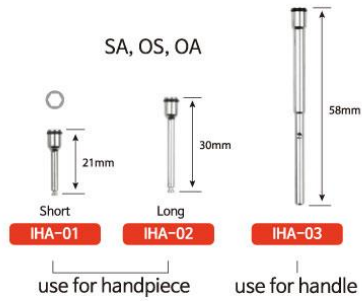
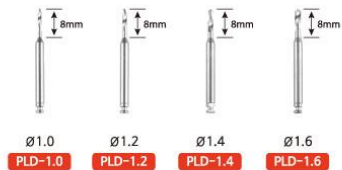
Short time for implant The application of all charge drills.
 Strong durability against heat.
 Simple to use multiple implant.

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